



Sleep Active Head Office
Tel No: (011) 609 8855
Tel No: (011) 609 8288
Website: www.sleepactive.co.za
Email: info@sleepactive.co.za
Fax No: 086 617 1121

SLEEP ACTIVE OVERLAY CARE INSTRUCTIONS

General Care Instructions

- **NEVER** wash your Sleep Active® overlay
- **NEVER** soak your Sleep Active® overlay
- **NEVER** clean you Sleep Active® overlay with any chemical detergents
- If you overlay is dirty, please use a warm damp cloth and gently wipe the mark/s away.

Cleaning spills

The Memory Foam® should never be wet, and never wash the Memory Foam®. Because the overlay should never be wet, we cannot promise that the following steps will completely remove the stain/s or odor/s. If you accidentally spill liquid on your overlay, please take the following steps immediately.

- Soak up as much of the liquid as possible using absorbent cloths.
- Once excess moisture is removed, let the overlay air dry. To speed up the drying process, place a fan so that it blows directly on the overlay.

Mattress Protectors

Please ensure that you use our recommended mattress protector as it is designed to protect your overlay without impacting the feel or performance.

Electric blankets or heating pads

We do not recommend using an electric blanket or heating pad with your overlay. Due to the temperature-sensitivity of the Memory Foam®, adding high heat can hinder its ability to react to your body temperature and conform to your body shape. If you ever do use an electric blanket or heating pad, we recommend limiting its use to 30 minutes, and then only on low or medium heat.

Transporting or storing

When transporting or storing your overlay, we recommend that you roll the overlay up, tie it and place in a plastic bag.